# INFRARED ROBOTIP

## Instructions for Use

<p>Model IRT101: Infrared RoboTip <span><span><span></span><span> </span></span><b>(EN)</b></span></p>
<p>Instructions for use – Infrared RoboTip:- Manufactured by Surgilüm LLC, 1717 Shipyard Blvd, Wilmington</p>

Thank you for purchasing your Infrared RoboTips. Please read these instructions carefully, in order to achieve the best possible results.

Indications for use
The RoboMarker® RMS101 and the Infrared RoboTip IRT101 serve in combination as a corneal marking system before ophthalmic surgeries and should be used only by trained healthcare personnel. The system places highly accurate marks on the cornea for alignment of toric intraocular lenses, limbal relaxing incisions, and refractive laser procedures

<p>Contraindications <b>DO NOT USE INFRARED ROBOTIPS WITH ANY PATIENT THAT HAS SENSITIVITY OR ALLERGY TO GENTIAN VIOLET (CRYSTAL VIOLET) DYE.</b></p>
---

<p>Precautions <ul style="list-style-type: none"><li>Use of this device is limited to health care professionals with appropriate training.</li> <li>Only use Infrared RoboTips for their intended use and as directed within these instructions.</li> <li>Single use Infrared RoboTips are sterile and are intended for single use ONLY. Do NOT attempt to re-sterilize.</li></ul></p>
--

<p>Compatibility The single use Infrared RoboTip is only compatible with the RoboMarker® that is produced by Surgilüm LLC. Do not attempt to use the Infrared RoboTips with other systems or devices.</p>
---

<p>CAUTION Single use Infrared RoboTips <b>CANNOT</b> be reused. Any attempt to do so may cause harm or injury</p>
--

Inserting a single use Infrared RoboTip into the RoboMaker®

- Open and remove the single use Infrared RoboTip holding it by the cap so as to not contaminate the inked marking edge.
- Insert the stem of the Infrared RoboTip into the aperture in the center of the dial.
- Locate 1 of the 2 indicators arrows on the single use Infrared RoboTip, and rotate the Infrared RoboTip until the indictor arrow aligns with the desired axis degree mark on the dial.
- Carefully remove the cap.

<p>Corneal Marking Procedure</p> <ol style="list-style-type: none"><li>If the femtosecond-laser is about to be performed: Use a firm one-second application since it is unlikely that the infrared blocking compound can be blinked off in just a few minutes.</li> <li>If the pre-op drops have just started and the patient is waiting for surgery: The Infrared RoboTip should be held on the eye for 3-5 seconds with a gentle finishing “RETROPULSION” of the eye, before removing the tip. The retropulsion should be relatively gentle; if too aggressive the cornea will receive a wide band of infrared blocking black ink. If the ink is too thick, it will block the femtosecond laser later.</li></ol>
--

Note: In all cases be sure to hold the eye open for 10 seconds to allow the ink to “set.”

<p>Reference RMS101 RoboMarker Handle - Instructions for Use for additional information.</p>
--

<p>Model IRT101: Infrared RoboTip <span><span><span></span><span> </span></span><b>(NL)</b></span></p>
<p>Gebruiksaanwijzing– Infrared RoboTip:- Manufactured by Surgilüm LLC, 1717 Shipyard Blvd, Wilmington</p>

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